# STANDARD *M10* Flu/RSV/SARS-CoV-2

STANDARD™ M10 Flu/RSV/SARS-CoV-2

## QUICK REFERENCE INSTRUCTIONS

For use with STANDARD™ M10 system

REF M10-CVFR-01

More Info

# QUICK REFERENCE INSTRUCTIONS (QRI)

Please read the complete Instruction for use (IFU) prior to use.

## Before you begin

- Store the STANDARD™ M10 Flu/RSV/SARS-CoV-2 cartridges at 2 ~ 28°C (36 ~ 82°F).
- This test is only for nasopharyngeal swab specimens.
- Always wear clean gloves and follow your institution's safety policy when handling patient samples.

Refer to IFU for Warnings and Precautions, Specimen Collection, Transport, and Storage, and Quality Control.

## TEST PROCEDURE

### I. Starting the STANDARD™ M10 system

#### 1. Turn on M10

Connect the M10 Console and Modules, then turn on each switch.


M10 Console  
(Left)

M10 Module  
(Back)

#### 2. Log in

Enter user name and password, and touch the Log in button.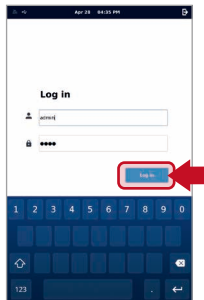

#### 3. Select M10 Module

Touch the button of an available Module on the Home screen.

Home screen

The status indicator will blink green, and the door opens automatically.

#### 4. Scan/Type Patient/Sample ID

Scan the Patient/Sample ID using the integrated barcode reader of the M10 Console. (or type in using a virtual keyboard)

Patient ID is optional and can be turned off in the 'SETTINGS'.

#### 5. Scan test Cartridge

Scan the Cartridge barcode using the integrated barcode reader of the M10 Console.

After scanning the cartridge, it automatically proceeds to the next step.

#### 6. Sample loading guide

Watch the animated guide for cartridge preparation and sample loading. Touch the screen to continue.


Sample Guide screen

## II. Loading a sample into the cartridge

- Start the test within 30 minutes of loading the sample into the STANDARD™ M10 Flu/RSV/SARS-CoV-2 cartridge.
- If the sample contains a lot of suspended matter or is highly viscous, as shown in the example on the right, it is recommended to use only the supernatant for testing after brief vortexing and centrifugation.

Specimen with suspended particles

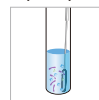Viscous specimen

#### 1. Remove Safety Clip

Remove the Safety Clip from the Cartridge.


#### 2. Pierce Cartridge

Press down on the lid to pierce the sealed cartridge.

#### 3. Transfer Sample

Open the lid and dispense 300 µL of the sample into the sample chamber.

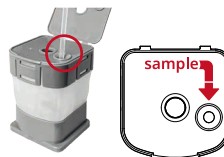#### 4. Close Cartridge lid

Close the cartridge lid.

### III. Running a test

#### 1. Load Cartridge into the Module

Place the cartridge into the assigned M10 Module and close the door manually.

The PCR chamber of the cartridge should face the inside of the M10 module.

#### 2. Confirm the test

After confirm the sample and cartridge information, touch the 'OK' button.

Confirm the test information before starting the run


Patient ID	51551551
Sample ID	21515151
Test Type	Specimen
Assay Name	Flu/RSV/SARS-CoV-2

Reset
OK

Confirm the Test screen

#### 3. Start Run

The remaining time is displayed on the M10 Console monitor.

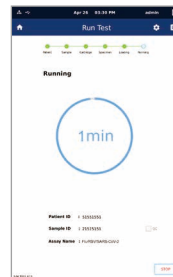Running screen

## RESULT INTERPRETATION

Result	Flu			RSV		SARS-CoV-2		
	Flu A	Flu B	IC	RSV	IC	ORF1ab	N	IC
Flu A Positive	+	-	+ / -	-	+	-	-	+
Flu B Positive	-	+	+ / -	-	+	-	-	+
RSV Positive	-	-	+	+	+ / -	-	-	+
SARS-CoV-2 Positive	-	-	+	-	+	+	-	+ / -
	-	-	+	-	+	-	+	+ / -
	-	-	+	-	+	+	+	+ / -
Negative	-	-	+	-	+	-	-	+
Invalid	-	-	-	-	-	-	-	-

- Each target (Flu A, Flu B, RSV, SARS-CoV-2) may have a co-infection.

- If the IC is negative and the results for any of the targets are positive, the results for all targets are considered valid. A high copy number of target-specific gene can lead to reduced or absent IC.

- If an invalid result is confirmed in one or more of the pathogen results, that tests will be invalidated. Please conduct a re-test.

Outcome (Home screen)	Result (Review screen)	Description
Positive	<b>+</b>	At least one pathogen/ target gene is positive.
Negative	<b>-</b>	No pathogen was detected.
Invalid	<b>!</b>	IC signal does not have a Ct within the valid range.
Error	<b>X</b>	The test failed because either an error occurred or the test was canceled by the user.

Outcome (Summary screen)	Result (Summary screen)	Description
IC Valid	<b>V</b>	IC has a Ct within the valid range. : The test was completed. Report positive/negative results of pathogens according to the interpretation shown in the console.
IC Invalid	<b>!</b>	IC does not have a Ct within the valid range. : Pathogens detected as positive shall be reported, but all negative results (tested but not detected) are invalid. Repeat the test.
IC Error	<b>X</b>	The test failed because either an error occurred or the test was canceled by the user. Repeat the test.

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